

NEURONIC

Thank you for purchasing the
Neuradiant 1070.


Fitting Guide

Remove the helmet and components from the case. Take the velcro strips from the adhesive paper and apply onto the inside of the helmet as shown in the diagram.

Apply the sponges or foam headpiece onto the velcro so the unit is fitted during use.

Do not place any velcro on the clear plastic inside the helmet.

Bottom View

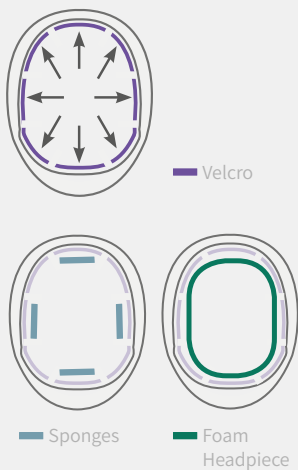


Turning on the Device

Connect the helmet with the controller, connect the controller with the first power adapter.

Connect the controller with the second power adapter.

After connecting the cables, you can turn on the device via the **button** on the **top right**.

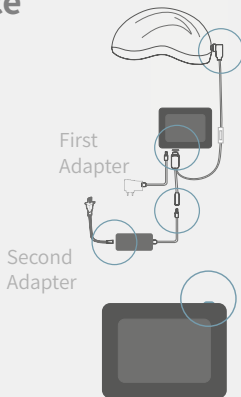


Selecting a Program

Select one of our 4 programs as shown above for general wellness.

Custom Program:

To be used when instructed by a clinician. Contact us at team@neuronic.online for more information.

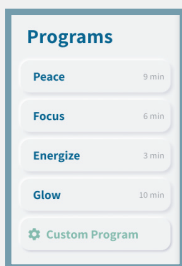


It's recommended to use it **morning and evening**, just like brushing your teeth!



Starting a Program

Now that you have selected your program, click the start button when the helmet is placed on your head.

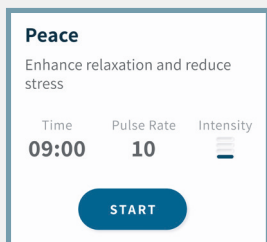

For more information about us, the device, the science and much more please visit our website at neuronic.online

Clinical Protocol Passcode:

Note: Only available with 4 quadrant device.

Our product cannot guarantee the same effects on each individual. There are no known side effects of using our device. Still, we advise you to consult us or your local doctor and conduct your independent research before using the device.

Please note this is not a medical device, we do not claim to treat, cure, prevent or diagnose diseases.